

## USB Switch Interface Box Installation Instructions

1. After you have turned on your computer, plug the USB Switch Box into the USB port on your computer. Next you will need to install a software driver that will communicate with the USB Switch Box. **Windows 2000 and XP** will install the driver automatically so that none of the following steps are required.

2. The operating system will detect the new hardware installation and “**Add New Hardware Wizard**” will automatically search new devices for USB Human Interface Device. Click “**Next**”



3. What do you want Windows® to do? Select “**Display a list of all the drivers in a specific location, so you can select the driver you want**” then Click “**Next**”




4. Select the radio button that says “**Show compatible hardware**” then select “**USB Human Interface Device**” under Models. Click “**Next**”



5. Windows is ready to install the driver now. Go ahead and Click “**Next**”



**5a.** If Windows is unable to locate the driver on your hard drive, you will see the screen directly below. Insert your Windows CD-ROM. Use the "**Copy files from**" box and the "**Browse**" function to tell Windows where the driver is located. Click "**OK**"

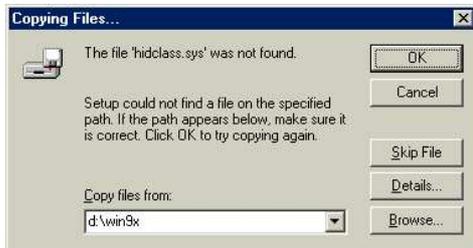

**6.** Windows has finished installing the driver. Click "**Finish**"



**7. To set up the switches,** turn to the *Scanning Selection* chapter in the **WiViK manual** and follow the directions. When you have selected the options you want for your switches, plug your switches into the Switch Interface Box. Single switch plugs are labeled 1-6; dual switch plugs are labeled 1-2 and 3-4; the joystick plug is labeled with a joystick icon.