

Switch Interface Cable Installation Instructions


1. After you have turned on your computer, plug the Switch Interface Cable (P.N. 13707) into the Joystick port on your computer. Next you will need to install a software driver that will communicate with the Switch Interface Cable.

2. Select the Windows® **Start** Menu, then **Settings**, then **Control Panel**. Once you have the Control Panel open, double click on the “**Game Controllers**” icon. For Windows ME and 2000 double click on “**Gaming Options**”

3. Next select “**Add**”



4. Select “**2-axis, 2-button joystick**” under Game Controllers then Click “**OK**”

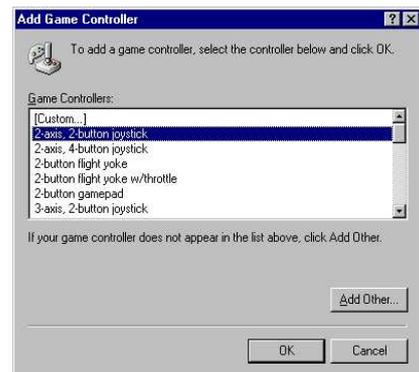

5. If everything is installed correctly, you should see the game controller you added and the status should read “**OK**”. Click “**OK**”



6. **To set up the switches**, turn to the *Scanning Selection* chapter in the **WiViK manual** and follow the directions. When you have selected the options you want for your switches, plug your switches into the Switch Interface Cable. You are ready to begin using WiViK.