

# Pathfinder Plus

## USB Flash Drive Software Update Instructions

 **Important!** Before performing an update: **Back up the vocabulary that is currently in your Pathfinder Plus.**

1. When the memory backup is completed, turn your Pathfinder OFF.
2. Plug your USB flash drive into the USB port on your computer.
3. Download the software update to your computer desktop.
4. Copy the file to your USB flash drive.
5. Once the file is copied, unplug the USB flash drive from your computer and plug it into the rectangular USB port on the left side of the Pathfinder.
6. Plug the battery charger into your Pathfinder.
7. Make sure the Pathfinder is OFF.
8. Press and hold the I/O and TOOLS keys at the same time. The Pathfinder screen will light. Release the I/O and TOOLS keys.
9. The update begins automatically. Watch the display screen for information.
10. When the update is finished, remove the USB drive from the Pathfinder.
11. Go to the Toolbox, open the **MAINTENANCE MENU** and select **Reset to Factory Defaults**. Select **OK** to exit the menu.
12. Load your saved vocabulary back into the device.