


# IntelliTools Activity Rows for Vanguard/Vantage

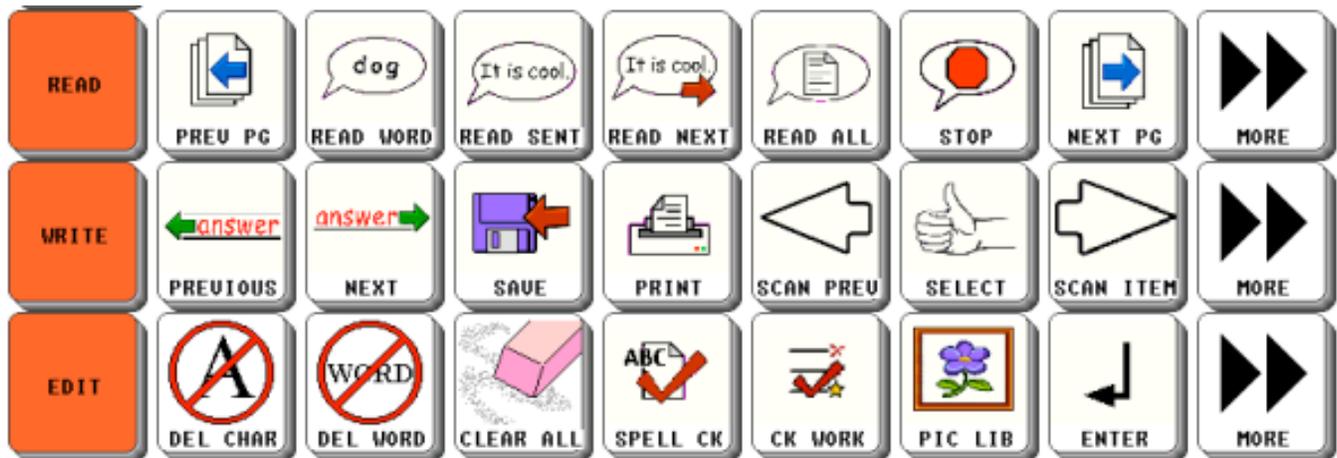

To load these rows into a Vanguard or Vantage, follow these steps:


1. Back-up your device before making major changes!
  - a. Download the ITOOLVT.MTI software from the PRC website and save to your hard drive.
  - b. Connect the device to the computer using the MTI cable.
  - c. Launch the MTI (Memory Transfer program) on your PC (AAC Keys must be off to run this program).
  - d. Check **Options** for **Device** settings.
  - e. Open Toolbox on the Vanguard/Vantage.
  - f. Select **Transfer Memory Menu**
  - g. Select **Save One User Area** and select the area you are currently using, just to have a back-up before you change anything. Give it a name including current date on your computer (i.e., JOHN121404).
2. Now you can bring in the IntelliTools Activity Rows.
  - a. Open the Toolbox on the Vanguard/Vantage and select **Transfer Memory Area**
  - b. **Load One User Area**
  - c. Choose the area you are currently using (the one you just backed up).
  - d. Select “No,” so vocabulary will be merged.
  - e. On the computer, highlight the saved ITOOLVT.MTI file.
3. You can go to Minspeak and use this file, but changing the user name is recommended.
  - a. From the Toolbox, select **Choose Overlay Menu**
  - b. **Rename User Area**
  - c. Select your user area (retitled IntelliTools), Clear Display, and enter the student’s name.

Troubleshooting:

If the above steps do not provide a merged user area, restore your previous user area using the **Load One User Area** option. Then you can load the ITOOLVT.MTI or ITOOLPF.MTI into a user area you are not using. Switch User Area to the IntelliTools user area and Copy Selected Activities (just the IntelliTools Activity) to your user. Switch back to your user and you will be good to go!

# IntelliTools Activity Rows for Pathfinder

The same activity rows will be used by the Pathfinder as on the previous page, as well as a fourth row:


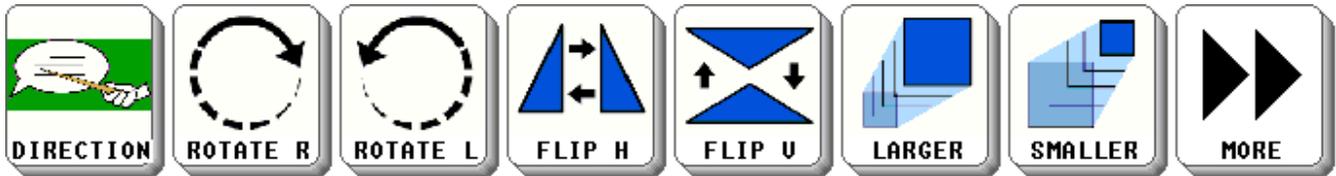


To load the IntelliTools Activity Rows into a Pathfinder, follow these steps:

1. Download the ITOOLSPF.MTI software from the PRC website and save to your hard drive.
2. Back-up your Pathfinder before making major changes!
  - a. Download the ITOOLSPF.MTI software from the PRC website and save to your hard drive.
  - b. Connect the device to the computer using the MTI cable.
  - c. Launch the MTI (Memory Transfer program) on your PC (AAC Keys must be off to run this program).
  - d. Check **Options** for **Device** settings.
  - e. Open **System Menus** on the Pathfinder
  - f. Select **Transfer Memory**
  - g. Select **Save One User Area** and select the area you are currently using, just to have a back-up before you change anything. Give it a name including current date on your computer (i.e., JOHN121404).
3. Now you can bring in the IntelliTools Activity Rows.
  - a. Open the **System Menus** on the Pathfinder and select **Transfer Memory**
  - b. **Load One User Area**
  - c. **Erase Old Vocabulary Before Loading**
  - d. Select a user area that you are not using, and confirm “Yes” you want to erase everything.
  - e. On the computer, highlight the saved ITOOLSPF.MTI file.
  - f. Select **OK** when finished.
4. Now you need to load this activity row into your user.
  - a. Open the **System Menus** and select **User Area**
  - b. Select **More Items**
  - c. Select **Copy Selected Activities** and then choose your user area.
  - d. Select the IntelliTools Classroom Suite icon, then **OK**
  - e. Confirm copy of selected activities from IntelliTools user to your user with **YES**
  - f. **OK**
  - g. Open the System Menus and select User Area
  - h. Switch User Area to your user.

IntelliTools Activity Rows will provide the user with a variety of functions in Classroom Suite.

In the first row, items are used for readback functions: Read Word, Read Sentence, Read Sentence and Advance, Read All. The “STOP” item stops speaking by activating the “escape” key.

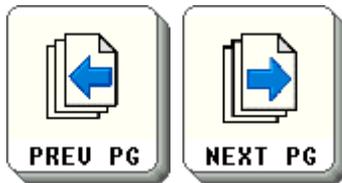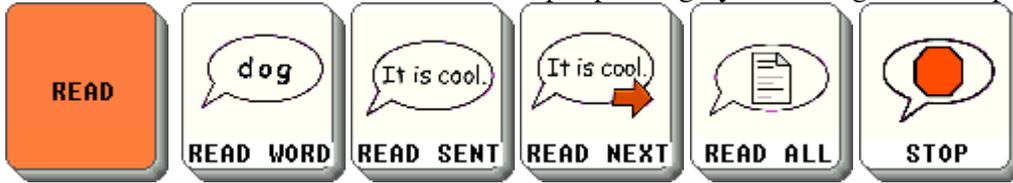

The other two items in this row are for navigation purposes and will provide the page up/down functions in word processing documents as well as turning pages in other files.

In the second row, items are grouped to be helpful during the writing process: Previous Answer Field, Next Answer Field (tab), Save, and Print:



The final three items are for selecting buttons on the computer screen. This activates scan tools in the IntelliTools Classroom Suite program. The user must repeatedly press “SCAN ITEM” to move forward (or “SCAN PREV” to move backwards) until the desired choice is highlighted, then press “SELECT” to activate.

In the third row, items are grouped for editing purposes: Delete Character, Delete Word, Clear All, Spell Check, Check Work, and Enter.



The “PIC LIB” item opens up the picture library in IntelliTools. The user can type a word and look for it in the picture library. To select the item, use the “Enter” key. The “ENTER” item is commonly used, and also works as a “Return” key.

The final row on the Pathfinder uses the F12 function for keyboard shortcuts. Classroom Suite accepts a keyboard sequence “F12 + the name of any button on the screen + enter” to activate buttons on the screen. This can also be done on the Vanguard/Vantage.

# Advanced IntelliTools Options – Not for the Meek!

Loaded with your files is an extra row called ICS TEMP that you can modify to meet your specific activity needs, as well as an ICS Draw page that works with Classroom Suite. To modify the activity row, begin by selecting buttons from an IntelliTools Classroom Suite activity toolbar that you would like to access with the Vanguard/Vantage. From within Classroom Suite, use control+click to reveal the button name. Then modify the ICS TEMP activity row on the Vanguard/Vantage by replacing the text “Button Name” with the name of the button. Leave the rest of the text:  
 (<SERIAL MARKER>=>F12.New Button Name<SERIAL MARKER><SERIAL MARKER>=>enter.)

The ICSDRAW page looks like this:



This page provides explore, design(draw), and paint functions in all IntelliTools Classroom Suite, IntelliPics Studio files. Don't forget to load the computer access pages (from the Overlay Menu on your Vanguard/Vantage) and link the MOUSE PG button to the appropriate page. The arrow keys move selected items on the page in very small increments, so be sure to watch carefully!