

ECO™2 USB Software Update

➡ Read the instructions before beginning the update.

You may want to print this page and then proceed to the update.

➡ To download and install this update you will need:

Your ECO2 and the ECO2 battery charger.

The system software you are currently using in your device should be a **lower version** than the version you want to download. To check your current version, go to the ECO2 Toolbox. Open the **MAINTENANCE MENU**. Look at the "System Information" box to check your Current Software Version.

On Your Computer

1. Go to www.prentrom.com
2. Click on "Downloads." Click on "ECO2." Click on "Software Updates".
3. Download the update you want to your computer desktop.
4. Plug the USB flash drive into the USB port on your computer.
5. Copy the update file to your USB flash drive.
6. Remove the USB flash drive from your computer and keep it nearby.

On Your ECO2

1. Plug your USB flash drive into a USB port on the right side of the ECO2.
- ➡ If you have made changes to your device vocabulary that you want to keep, use the **TRANSFER MEMORY MENU** in the ECO2 Toolbox to save the vocabulary to your USB flash drive before you start the update.
2. Turn **On** the ECO2 and plug in the battery charger.
3. Go to the Toolbox and open the **MAINTENANCE MENU**.
4. Select the **Update Software** option. Answer **YES**. Make sure the battery charger is plugged in.
5. When the update is finished, remove the USB flash drive from the device.
6. If you saved vocabulary earlier, use the **TRANSFER MEMORY MENU** to load it back into the device now.
- ➡ **Note:** If you load old vocabulary, you will **lose** any MAP fixes or MAP changes that came with the new software.
7. Remove the USB drive from your device and put it in a safe place. You may also unplug the battery charger if you wish.