

## **Unidad español 84 por ECO2**

These are the instructions that you need to get started with Spanish for ECO2. Please read them first. The memory transfer file will go into an ECO2 with software version number 2.06 or higher.

We hope you enjoy the Spanish program. ¡Buena suerte!

### **Transfer the memory**

1. The file you downloaded from PRC's web site is a zipped file. Click and drag the mti file (memory transfer) onto your desktop. Insert a USB drive into your computer port, then click and drag the mti file onto the USB drive's window.
2. Remove the USB drive from your computer and put it into your ECO2 USB port.
3. In the ToolBox of your device go to Transfer memory menu.
4. Select Load One User Area.
5. Select a user area that you are not using. The memory transfer will erase everything in the area you chose, and will put the Spanish program in.
6. When you see the prompt on the ECO screen, select the file named  
ECO2 Unidad esp 84 11\_29\_10.mti
7. Select OK to finish, then Go to Home.

### **Make a link between your new Spanish area and your custom area.**

1. Find the orange key in the device tools activity row that says Go to English 84.
2. Select the Set-up key, and then the orange key.
3. Select Spell Message or Define Key Function.
4. Select Clear Display.
5. Select Insert Tool and then Switch User Area.
6. Select your user area.
7. Change the label on the key if you want.
8. Select OK, and OK again.
9. Use the orange key to go to your usual area.
10. Find a key in the device tools activity row where you can make a link.
11. Use the directions above to make a link back to your new Spanish user area.

### **Choose a voice for your Spanish area.**

1. The American Spanish voices for ECO are RealSpeak Javier and Paulina, and Acapela Rosa. There are two other Acapela Spanish voices, Antonio and Maria. They speak European Spanish.
2. In the ECO ToolBox go to the Speech Menu. Select the synthesizer, then the voice.
3. **Important!** If you don't see the Spanish voices in your Speech Menu, you have to load them in from the CD that came when you received your device. These CD's are mailed separately from the device. If you need help finding or loading a Spanish voice, call PRC service at **1-800-262-1990**.

**Try speaking.**

1. Start with some interjections.
2. Then try some whole sentences, beginning with the sentence key.
3. The vocabulary manual for Spanish 84 is one of the documents that you downloaded. Print it out and use it to become familiar with the Spanish grammar structure.

**Word prediction**

There is not a separate Spanish word prediction dictionary for the ECO2. You may add Spanish words to your word prediction dictionary by going to the Word Prediction menu. Because it is an English dictionary, you may not add words with accented characters.

**Dictionary Menu**

There is only one speech dictionary in your ECO2. You may find some words in Spanish that are not pronounced correctly. You can add a phonetic pronunciation for these words in the Dictionary Menu

1. Go to the ToolBox and select Dictionary Menu.
2. Select Add a Word
3. Enter the proper spelling of the word, then select OK.
4. Enter a phonetic spelling of the word.
5. Select the text area to listen to it.
6. If you like the pronunciation, select OK.
7. If you don't like it, edit the phonetic spelling. You can add a space, an accent mark, or quotes to change the emphasis.

You may find that by correcting a word in Spanish, it gets worse in English. Try using digitized (recorded) speech to change either the Spanish or the English pronunciation.