# VTL Spanish Vocabulary Manual

---

Southwest American Spanish Bilingual  
Vocabulary for Vantage Lite and Vanguard

Vantage Lite, ECO, Vantage, Vanguard II, Vanguard, Pathfinder, SpringBoard, Language Activity Monitor (LAM), and Memory Transfer Interface (MTI) are trademarks of Prentke Romich Company.

DECTalk is a product of the Digital Electronics Corporation.

Minspeak is a registered trademark of Semantic Compaction Systems.

Minspeak Application Programs (MAPs) are trademarks for products manufactured by the Prentke Romich Company.

Unity is a registered trademark in the USA of Semantic Compaction Systems.

Windows, Windows XP, Windows Explorer, Windows Media Player, Microsoft Outlook, Microsoft Word, Windows WordPad and Microsoft PowerPoint are registered trademarks of Microsoft, Inc.

IntelliTools is a registered trademark of IntelliTools, Inc.

Natural speech supplied by: Fonix-DECTalk Text-to-Speech software, a copyright of Fonix Corporation, all rights reserved; and RealSpeak Text-to-Speech, a product of Nuance Communications, Inc.

Non-English text-to-speech supplied by the Acapela Group and RealSpeak Text-to-Speech, a product of Nuance Communications, Inc.

Computer access pages for Kurzweil, WYNN, TestTalker, Microsoft Word, and Microsoft Outlook created by Brian Simms, M.A. CCC-SLP, Augmentative Communication Specialist, Colorado Therapy Associates

WYNN and TestTalker are trademarks of Freedom Scientific, Inc. Kurzweil 3000 is a trademark of Kurzweil Educational Systems, Inc.

Bluetooth Wireless Technology is a registered trademark of Bluetooth SIG, Inc. The Bluetooth word mark and logos are owned by Bluetooth SIG, Inc.

October, 2008

## **PRENTKE ROMICH COMPANY**

1022 Heyl Rd. • Wooster, Ohio 44691

Telephone: (800) 262-1984

Fax: (330) 263-4829

E-Mail Address: [info@prentrom.com](mailto:info@prentrom.com)

Web Site Address: [www.prentrom.com](http://www.prentrom.com)

## **Prentke Romich International**

PRC House, Minerva Business Park  
Lynch Wood, Peterborough, Cambridgeshire

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## **Preface**

### A Note from the Author

The Bilingual Vantage Lite has been brought to you by a team of people who believe that the language of the home and of one's family is as important as the language which one uses at school or work. We also believe that AAC users, like others, are capable of learning and benefiting from becoming bilingual. Some users may communicate in Spanish only at home while others may use Spanish in all aspects of their daily life.

Spanish is a rich language whose vocabulary can vary even from town to town. The vocabulary in the Bilingual Vantage Lite is based on Sonoran Spanish used in northern Mexico and in many areas of the Southwestern United States. We have also consulted people from El Salvador and Guatemala and have sought to include vocabulary items which would be understandable to them as well. One of the vocabulary testers was from Ecuador, while most were Mexican-American. We encourage you to customize vocabulary based upon your community and your preferences.

Using your Bilingual Vantage Lite, you will be able to learn to express needs, feelings and ideas, to ask questions, to speak with all your relatives, to participate in worship, and to converse in Spanish or English with people in stores, at school and in medical offices. You can even take a Spanish class at school, You can also import photos and store messages to create a family album or talk about your pets or your vacation.

### Acknowledgements:

The following people helped us in our search for words and in our vocabulary testing so that this device could become a reality:

Sr. Veronica Mendez Tabanico and  
Carmen Leticia Robles Gonzalez of the Casa Hoga Nuestros  
Hermanitos, Nogales, Sonora;

The families of the St. Andrew's Children's Clinic in Nogales;

Lourdes Garino, autism teacher; Eric Gonzalez and Juan Lopez,  
students, of the Sunnyside Unified School District, Tucson, AZ,,

and other friends and Spanish-speaking family members who gave  
feedback and encouragement.

We can never thank them enough.

For Lupita  
And Bryan

## Introduction to Unity Español 15

The Unity Español 15 is based on the use of single words to communicate. In the American Spanish Bi-lingual program for Vantage Lite, this 15 location program has fewer choices and larger keys than the other user areas.

There are 13 keys with basic vocabulary words and phrases. Some of these words open the top row of keys. For example, if you select the key labeled “me gusta” you see that the top row contains choices. These choices can be customized. Refer to your Operator’s Manual on using the set-up key.

**NOTE:** There are no true activity rows in this program such as there are in the other two user areas. You can modify any key that you see by using the set-up key, which is the #2 key on the device frame. You cannot modify the top row of keys by selecting Modify Activity in the ToolBox.

You can say just a word at a time, or put together two and three word phrases and sentences such as “I want more.”, “Look at me.”, and “Don’t touch that.”

Switching to English:

1. Select the ELEGIR key.
2. Select the SI NO key.
3. Select the key labeled ENGLISH UNITY 15.

When you choose the key labeled ACTIVITY you will see the **Clear Display**, and keys at the top for SI NO, SOBRE ME, SALUDOS, and EMOCIONES.

When you choose the CATEGORIAS KEY, you will see links to categories and other pages.

Navigation: On all pages, the top right corner key labeled REGRESAR, goes back to the main page of 13 basic words. When just the top row of keys changes, use the blue arrow labeled REGRESAR.

## Introduction to English Unity 15

The English Unity 15 is based on the use of single words to communicate. In the Southwestern American Spanish Bilingual program for Vantage Lite, this 15 location program has fewer choices and larger keys than the other user areas.

There are 13 basic vocabulary words. Some of these words open the top row of keys. For example, if you select the word “like” you see that the top row contains choices. These choices can be customized. Refer to your Operator’s Manual on using the set-up key.

**NOTE:** There are no true activity rows in this program such as there are in the other two user areas. You can modify any key that you see by using the set-up key, which is the #2 key on the device frame. You cannot modify the top row of keys by selecting Modify Activity in the ToolBox.

You can say just a word at a time, or put together two and three word phrases and sentences such as “I want more.”, “Look at me.”, and “Don’t touch that.”

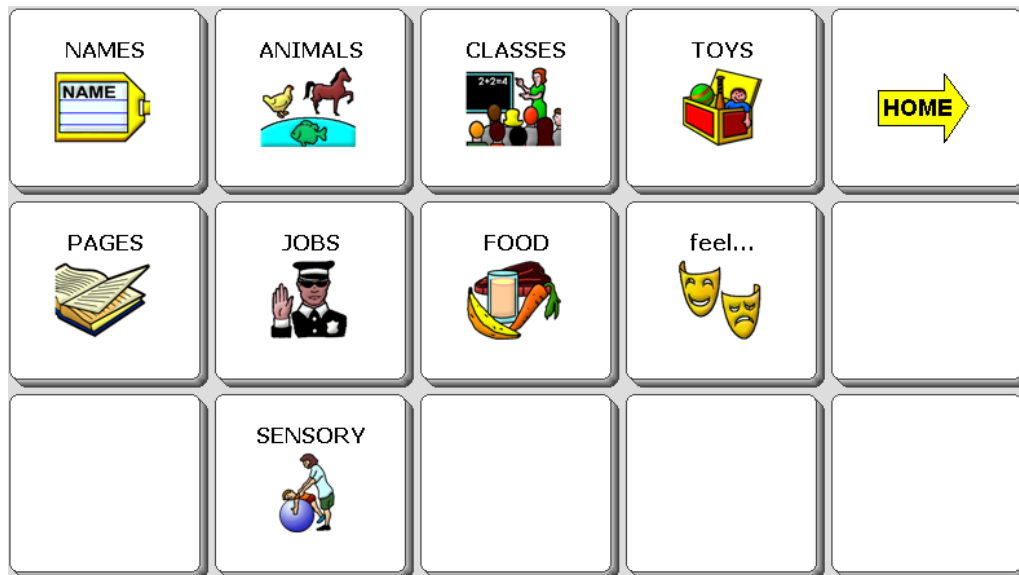
Switching to Spanish:

1. Select the CHOICES key.
2. Select the YES NO key.
3. Select the key labeled SPANISH UNITY 15.

When you choose the key labeled **ACTIVITY** you will see the **Clear Display**, and keys at the top for YES NO, ME, GREETINGS, and I feel....

When you choose the **CATEGORIES** KEY, you will see links to categories and other pages.

Navigation: On all pages, the top right corner key labeled **HOME**, goes back to the main page of 13 basic words. When just the top row of keys changes, use the blue arrow labeled **GO BACK**

## Introduction to the Puente 45

### Overlay

The structure of the Puente (Bridge) MAP provides an introduction to Word ending and Word order skills. This MAP is not a transliteration of the American English Unity 45 MAP. It is its own representation of the Spanish language as spoken in Northern Mexico and the American southwest. The Puente is an intermediate step between the word-by-word translation of SpringBoard Bilingual Spanish and the Unity 84 Español MAP. For some users the Puente will be an end in itself, while for others, it will live up to its name and facilitate the transition to real generative language provided by the Unity 84 Español vocabulary.

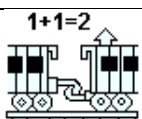
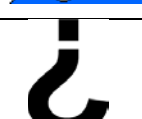
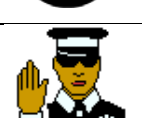
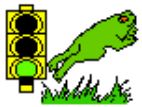
The top row of nine keys is called the main Activity Row. You can scroll through the activities in the row by selecting the ACTIVIDAD key. This top row of nine keys is also the place where you can see Dynamic Activities appear after you select certain keys in the four rows below. For example, if you select the PERRO (DOG) icon, you will see choice of animals on the top row.

The bottom four rows are called Core Vocabulary. Words in Core are used very frequently by most people. These words are organized by categories, for example all the food words begin with the PAN (BREAD) icon.

At the top right corner you can find the key for the Pages page. Many Fringe Vocabulary words are located on pages. For example, there is a Religion Page on the Pages page. Not everyone will use these words every day. That is why they are called Fringe Vocabulary.

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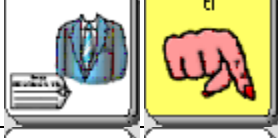
## Grammatical Structure

The Puente MAP is based on the following grammatical structure:

### Pronouns

Sequence	Word	English meaning
	yo	I
	mi	me
	mí	me
	mío	mine
	tú	you (informal)
	tu	you
	tí	you
	tuyo	yours

## Verbs

There are 24 verbs, each represented by a category icon on the vocabulario nucleo (core vocabulary) display. Infinitives (comer, ir, leer) may be accessed using the category icon + monito de acción (Mr. Action Man).

Secuence	Verb	English meaning
 	<i>llevar</i>	to wear
 	<i>gustar</i>	to like
 	<i>trabajar</i>	to work
 	<i>preguntar</i>	to ask
 	<i>jugar</i>	to play
 	<i>ser</i>	to be (stage of existence)
 	<i>tener</i>	to have
 	<i>sentir</i>	to feel
 	<i>leer</i>	to read
 	<i>escuchar</i>	to listen to, to hear

Sequence		Verb	English meaning
		<i>terminar</i>	to stop, to finish
		<i>querer</i>	to want
		<i>venir</i>	to come
		<i>ir</i>	to go
		<i>ganar</i>	to get, to gain, to win
		<i>estar</i>	to be (requires an adjective following)
		<i>pintar</i>	to paint
		<i>ayudar</i>	to help
		<i>comer</i>	to eat
		<i>hacer</i>	to make
		<i>necesitar</i>	to need

Sequence	Verb	English meaning
 	<i>tomar</i>	to drink
 	<i>dormir</i>	to sleep

As there is no need for transposition of word order for questions in Spanish and as the unity 84 Español MAP will require a third icon to mark gender and number for verb conjugation, the word order for pronoun-verb constructions is reversed. That is, rather than beginning with the pronoun as is done in the English Unity MAPs, these structures begin with the category icon, and end in “yo” or “tú”. Using the Puente MAP, it is possible to conjugate first and second person forms for the present indicative tense for each of the verbs in the verb infinitives table above.

### Verb conjugation examples

Sequence	Word	English meaning
 	<i>tomo</i>	I drink
 	<i>tomas</i>	you drink
 	<i>como</i>	I eat
 	<i>comes</i>	you eat
 	<i>tengo</i>	I have
 	<i> tienes</i>	you have

This provides a starting point for the three-icon sequences which will be needed for conjugating verbs for all but the present indicative (“yo tomo”) and infinitives (“tomar”) when the person transitions to the Unity 84 Español MAP.

It should be noted that shortcuts are provided for two reflexive verbs, “gustar” and “sentir”

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## Datives, Articles, and Determiners

The MAGO (Determiner) icon precedes determiners. Here are some examples:

Sequence	Word	English meaning
  eso	<i>eso</i>	that
  este	<i>este</i>	this
  esta	<i>esta</i>	this
  ésto	<i>ésto</i>	
  todas	<i>todos</i>	all

Sequence	Word	English meaning
 	<i>ninguno</i>	none
 	<i>cada</i>	each

## Interjections

Interjections begin with the PIROTECNICAS icon. Here are some examples:

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## Questions

Questions may be accessed using the grammar icon PREGUNTA followed by the corresponding category icon. Here are some examples:

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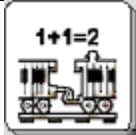
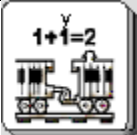
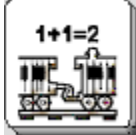
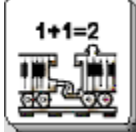
## Prepositions

The grammar icon PUENTE precedes prepositions. PUENTE is a combined icon on a key with the grammar icon BROCHA. Here are some examples of prepositions:

Sequence	Word	English meaning
 	<i>con</i>	with
 	<i>sin</i>	without
 	<i>sobre</i>	over

## Conjunctions

Conjunctions are represented beginning with the grammar icon TREN. Here are some examples:

Sequence	Word	English meaning
 	<i>y</i>	and
 	<i>o</i>	or
 	<i>pero</i>	but

## Adjectives

Adjectives are usually a three icon sequence consisting of the category icon + BROCHA + gender ending. Here are some examples:

Sequence				Word	English meaning
				bonita	pretty
				bonito	pretty
				mucha	much
				mucho	much

## Adjectives in activity rows

Some adjectives are store in dynamic activity rows. Here are some examples:

Sequence	Activity row	Word examples
	EMOCIONES	<i>feliz, triste, emocionado, enfadado, enojado, asustado, lastimado</i>
	COLORES	<i>rojo, anaranjado, amarillo, verde, azul, morado, rosa</i>

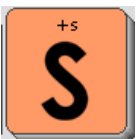
## Nouns

A category icon + the ALTO icon (which becomes a concrete picture) will speak a noun.

Here are some examples:

Sequence	Word	English meaning
 	<i>ropa</i>	clothing
 	<i>día</i>	day
 	<i>permiso</i>	permit, permission
 	<i>pregunta</i>	question
 	<i>juego</i>	game
 	<i>familia</i>	family
 	<i>dinero</i>	money
 	<i>libro</i>	book
 	<i>música</i>	music
 	<i>cosa</i>	thing

**Noun plurals:** You can make many nouns plural by adding 2 hits on the ALTO icon.

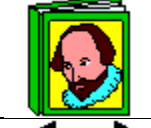
Sequence	Word	Significado en ingles
  	<i>fruta</i>	fruit
   +  	<i>frutas</i>	fruits

Notice that the plural form of the noun appears below the singular form with the icon for Mother Hubbard with lots of things in her cupboard. Some nouns do not have a plural form.

Sequence	Word	Significado en ingles
 	<i>libros</i>	books
 	<i>familias</i>	families

## Nouns in activity rows

There are many nouns stored in dynamic activity rows. These are called fringe vocabulary because they are needed for certain situations and activities, but are not commonly used every day all the time. Here is a list of the category icons that have dynamic rows linked to them.

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## Activity Rows in Punto 45

There are many activity rows in the Punto MAP. Most of these are hidden activities that appear as dynamic activities. A dynamic activity is one that you see when you select a key or sequence with a link to that particular activity. These activity rows may be in a slightly different order in your device.

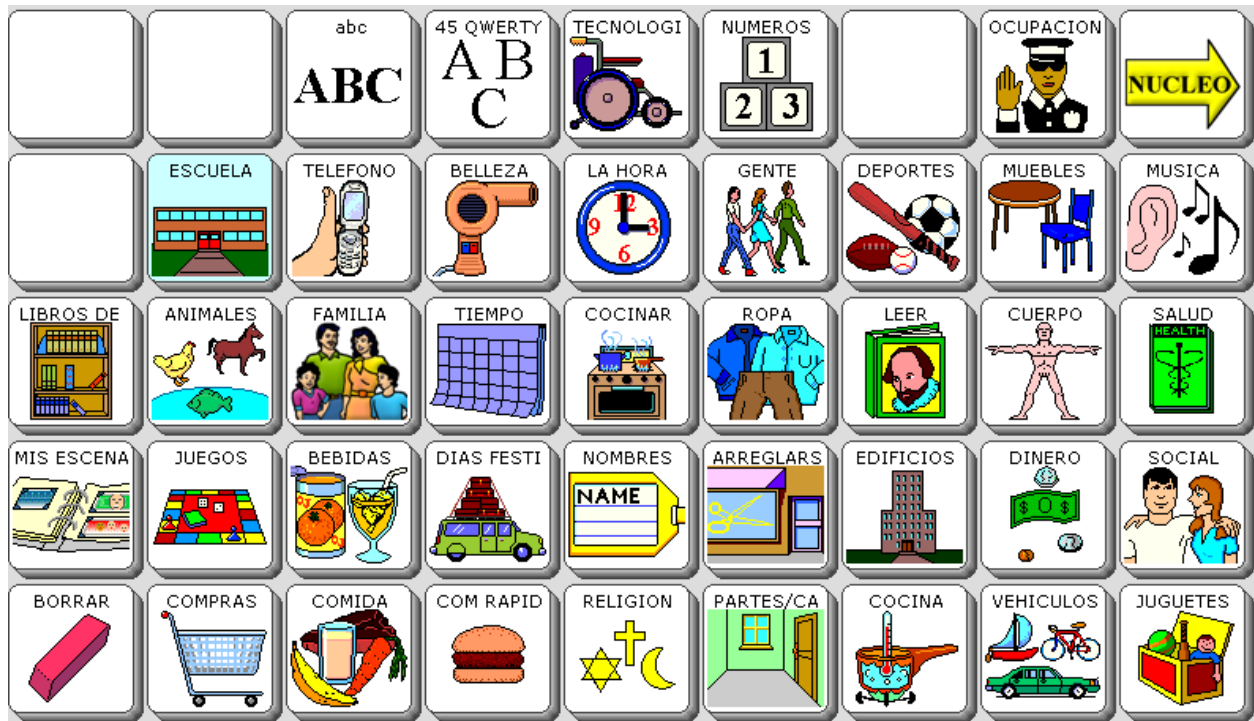
## Activity Rows in Punto 45 con't.

## Pages in Puente 45

You can use the Pages key at the top right corner to go to the Pages page. These pages contain fringe vocabulary, that is, vocabulary for specific occasions and environments.

There are other pages in the Puente MAP that are not linked to the Pages page. You can find these pages in the ToolBox by going to the Page Menu.

## Introduction to the English Bridge 45

### Overlay

The structure of the English Bridge MAP provides an introduction to word ending and word order skills. This MAP is not the same as American English Unity 45 MAP. The English Bridge was written to reflect as closely as possible the Spanish Puente 45. For some users the Bridge will be an end in itself, while for others, it will live up to its name and facilitate the transition to real generative language provided by the Unity 84 Sequenced vocabulary. All the other MAPs for American English Unity are available from the User Area menu when you select Replace User Area and go to Unity.

The top row of nine keys is called the main Activity Row. You can scroll through the activities in the row by selecting the ACTIVITY key. This top row of nine keys is also the place where you can see Dynamic Activities appear after you select certain keys in the four rows below. For example, if you select the DOG icon, you will see choice of animals on the top row.

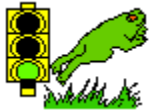
The bottom four rows are called Core Vocabulary. Words in Core are used very frequently by most people. These words are organized by categories, for example all the food words begin with the APPLE icon.

At the top right corner you can find the key for the Pages page. Many Fringe Vocabulary words are located on pages. For example, there is a Religion Page on the Pages page. Not everyone will use these words every day. That is why they are called Fringe Vocabulary.

## Icons

Icon	Name	Category(ies)	Word Examples*
	NAME/SUIT	Clothing Names	<i>coat, hat, jacket name</i>
	MR. ACTION MAN	Verbs	<i>Extra Verb Row – Medical Verbs, Shoe Verbs, Hand Verbs, Thinking Verbs, Face Verbs, ...</i>
	SUN	Like	<i>like, funny, day</i>
	INTERJECTIONS	Interjections	<i>thank you, sure, all done, all gone</i>
	DETERMINER	Determiners	<i>this, that, all, each, none</i>
	CONJUNCTIONS	Conjunctions and numbers	<i>and, because, but, until, or</i>
	PREPOSITIONS/ADJ	Prepositions and adjectives	<i>to, up, beside, with</i>
	QUESTION	Question words	<i>ask, who, what, when, where, how old, how</i>
	POLICE	Work and occupations	<i>work, teacher, aide, doctor, O.T.</i>
	ACTIVITY	This tool scrolls through the activity rows	
	I	Pronoun I	<i>I, me, my</i>

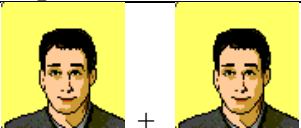
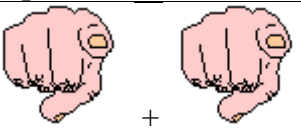
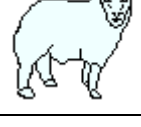
Icon	Name	Category(ies)	Word Examples*
	DICE	Games	<i>play, match, dice, game.</i>
	FAMILY	Family	<i>mom, dad, grandma, grandpa</i>
	MONEY	Money	<i>money, have, change, penny, nickel, dime, quarter</i>
	MASKS	Feelings	<i>feelings, happy, sad, bored, mad, sick</i>
	BOOK	Books	<i>read, favorite, notebook, story, magazine</i>
	MORE MUSIC	Music	<i>more, music, hear, song, album, radio, drum, cymbal, piano, guitar</i>
	STOP	Stop, endings, and adverbs	<i>stop, not, just, ready, away</i>
	DELETE LAST SELECTION	Deletes the last thing you chose	
	YOU	Pronoun You	<i>you</i>
	WANT	People	<i>want, person, people, friend, kid, man, men, woman, women, boy</i>
	DOG	Animals	<i>come, animal, pets, farm animals, zoo animals, insects, birds, baby animals</i>

Icon	Name	Category(ies)	Word Examples*
	FROG	Go	go, long
	THUMBS UP	Positives	<i>to get, good</i>
	HOUSE	Buildings	to live, house, home, home parts, rooms in the house
	RAINBOW	Colors	paint, color, pretty, red, yellow, blue, green, purple
	BATH	Body parts and health	help, clean, head, face, arm, leg, eye, ear, nose
	IT	Pronoun It	it, it is, it will, it can
	NO	Negative things	don't
	EYE	Look	look, see, thing, things
	APPLE	Food	eat, to eat, breakfast, lunch, apple, banana, grapes, pretzel, popcorn, chips
	GOD	God, holidays	God, make, holidays and holiday greetings
	UMBRELLA	Weather	need, weather, bad, sun, cloud, wind, rain, fog, snow
	JUICE	Drinks	drink, thirsty, tea, coffee, water, milk
	BED	Furniture	sleep, bed, table, chair, tired

## Grammatical Structure

The English Bridge MAP is based on the following grammatical structure:

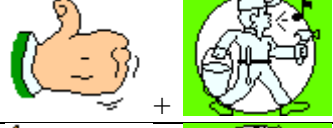
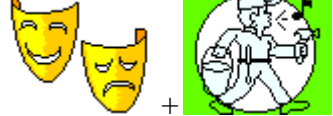
### Pronouns

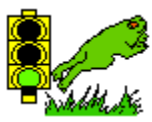
Sequence	Word	Sequence	Word
 +	I	 +	you
 +  +	me	 +  +	your
 +  +	my	 +  +	yours
 +  +	mine	 +  +	its
 +	it		

### Verbs

There are 25 two-hit verbs (root words), each represented by a category icon on the core vocabulary display. In the English Bridge it is possible to include more verb endings than in Spanish.

Sequence	Verb	Spanish meaning
 + 	ask	preguntar
 + 	dress	vestir
 + 	play	jugar

Sequence	Verb	Spanish meaning
	have	tener
	read	leer
	hear	escuchar
	look	mirar
	work	trabajar
	want	querer
	come	venir
	stop	terminar
	like	gustar
	get	ganar
	feel	sentir

Sequence	Verb	Spanish meaning
 + 	go	ir
 + 	live	
 + 	paint	pintar
 + 	help	ayudar
 + 	eat	comer
 + 	make	hacer
 + 	need	necesitar
 + 	drink	tomar
 + 	sleep	dormir

## Verb Conjugations in English Bridge 45

If you select the APPLE icon first, which represents the category of food, you can see the verb “to eat” has all forms of the verb showing.

The graphic below is a good illustration of the color coding in this program. All verb keys are colored green, the adjective key is blue, noun keys and noun keys are orange. The top row has changed to show the dynamic activity with subcategories of food.

## Verb phrases

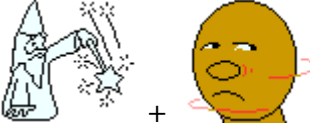
When you select the pronoun “I” you will see verb phrases. These are short-cuts which may help to save keystrokes.

## Determiners

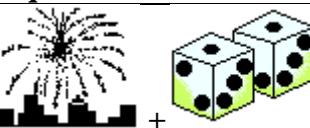
The DETERMINER icon precedes determiners. Here are some examples:

Sequence	Word	Spanish word
	this	eso
	that	ésto
	all	todos
	none	ninguno
	each	cada

The DETERMINER icon also begins the indefinite pronouns, like anybody, everybody, somebody, nobody.

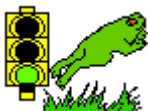
## Interjections

Interjections begin with the PIROTECNICAS icon. Here are some examples:

Sequence	Interjection	Spanish meaning
	I win!	¡Gané!
	How pretty!	¡Qué bonito!

## Questions

Questions may be accessed using the grammar icon QUESTION followed by the corresponding category icon. Here are some examples:

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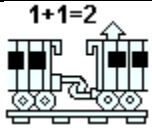
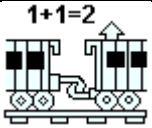
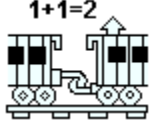
## Prepositions

The grammar icon PREP/ADJ precedes prepositions. PREP/ADJ is a combined icon on a key with the grammar icons of a bridge and a paintbrush. . The bridge is a first hit for prepositions because you cross “over” a bridge. Water flows “under” a bridge. Here are some examples of prepositions:

Sequence	Preposition	Spanish
 + 	with	con
 + 	without	sin
 + 	over	sobre

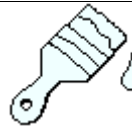
## Conjunctions

Conjunctions are represented beginning with the grammar icon CONJ.. Here are some examples:

Sequence	Word	English meaning
 + 	y	and
 + 	o	or

## Adjectives

Adjectives consist of the category icon + the ADJ paintbrush. Here are some examples:

Sequence	Adjective	Spanish meaning
 + 	pretty	bonita, bonito
 + 	much	mucha, mucho

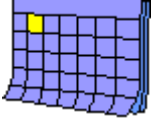
## Adjectives in activity rows

Some adjectives are store in dynamic activity rows. Here are some examples:

Sequence	Activity row	Word examples	Spanish
	FEELINGS	happy, sad, excited, bored, mad, scared, hurt	<i>feliz, triste, emocinado, enfadado, enojado, asustado, lastimado</i>
	COLORS	red, orange, yellow, green, blue, purple, pink	<i>rojo, anaranjado, amarillo, verde, azul, morado, rosa</i>

## Nouns

A category icon + the STOP icon (which becomes a concrete picture or Mother Hubbard) will speak a noun. Here are some examples:

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Notice that the plural form of the noun appears below the singular form with the icon for Mother Hubbard with lots of things in her cupboard. Some nouns do not have a plural form.

**Noun plurals:** You can make many nouns plural by adding 2 hits on the STOP icon.

## Nouns in activity rows

There are many nouns stored in dynamic activity rows. These are called fringe vocabulary because they are needed for certain situations and activities, but are not commonly used every day all the time. Here is a list of the category icons that have dynamic rows linked to them.

Category Icon	Activity Row	Example words
	NAMES	name
	CLOTHING	hat, jacket, shoe, sock
	GAMES	Mr. Potato Head, Uno, Bingo, video game
	FAMILY	mom, dad, grandma, grandpa
	MONEY	change, penny, nickel, dime, quarter, dollar, check
	READING	notebook, journal, magazine, story, TV guide
	MUSIC	album, radio, tape recorder, DVD, CD player, MP3 player
	ANIMALS	sub-categories of animas, PETS, FARM ANIMALS, ZOO ANIMALS,
	ROOMS	home parts and rooms – living room, kitchen, bathroom,
	BODY PARTS	sub-categories of body parts,
	NO	don't

Category Icon	Activity Row	Example words
	COMIDA	sub-categories of foods, DESAYNO, ALMUERZO, VERDURAS, FRUTAS, CARNE, ITALIAN, BOCADILLO
	HOLIDAY WORDS	sub-categories of holiday words
	WEATHER	sun, cloud, wind, rain, fog, snow
	DRINKS	tea, coffee, milk, water, juice
	OCCUPATIONS	teacher, assistant, doctor, O.T., P.T/
	FURNITURE	bed, chair, desk, table, couch, wheelchair

## Activity Rows in the English Bridge 45

There are many activity rows in the Puente MAP. Most of these are hidden activities that appear as dynamic activities. A dynamic activity is one that you see when you select a key or sequence with a link to that particular activity. These activity rows may be in a slightly different order in your device.

## Activity Rows in the English Bridge 45 con't.

## Pages in the English Bridge 45

You can use the Pages key at the top right corner to go to the Pages page. These pages contain fringe vocabulary, that is, vocabulary for specific occasions and environments.

There are other pages in the English Bridge MAP that are not linked to the Pages page. You can find these pages in the ToolBox by going to the Page Menu.

## Practice Sentences

<b>My house is your house.</b>									
									
<b>When are you going to play outside?</b>									
									
<b>Who wants to eat popcorn?</b>									
									
<b>Where is the hospital?</b>									
									
<b>How is your family?</b>									
									
<b>I have to go to the gas station.</b>									
									
<b>How many dogs do you have?</b>									
									

## Introduction to Spanish Unity 84

## Overlay

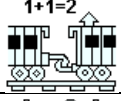
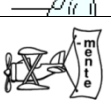
The structure of Unity 84 Español is simple and easy to learn. By combining the capabilities of this program, including core vocabulary, activity rows, and spelling with word prediction, the individual will have tools for expressing his or her ideas by generating original sentences when communicating in Spanish instead of depending on scripts which have been prepared by others.

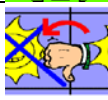
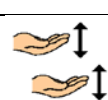
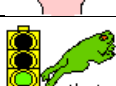
The top row of twelve keys is called the main Activity Row. You can scroll through the activities in the row by selecting the RETURN OBJ (REGALO)  key. This top row of twelve keys is also the place where you can see Dynamic Activities appear after you select certain keys in the four rows below. For example, if you select the DOG (PERRO) icon, you will see choice of animals on the top row.

The bottom six rows is where Core Vocabulary is stored. Words in Core are used very frequently by most people. These words are organized by categories, for example all the food words begin with the BREAD (PAN) icon.

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Icon	Name	Category	Word examples
	A	single hit for a	<i>a</i>
	MONITO DE ACCION	VERBS	<i>lo</i> , VERB ACTIVITY ROWS
	VERB+NDO	GERUNDS	<i>el</i> , Verb ending for gerund, present participle
	VERB P_PAS	PAST TENSE VERBS	<i>la</i> , Verb ending for past preterite
	VERB + ADO/IDO	PAST PARTICIPLE VERBS	<i>los</i> , Verb ending for past participle
	VERB	PAST IMPERFECT VERBS	<i>las</i> , Verb ending for past imperfect
	PULGAR ABOJO	PAST CONDITIONAL VERBS	Verb ending for past conditional <i>quitar, malo</i>
	MANOS	PRESENT SUBJUNCTIVE VERBS	Verb ending for present subjunctive
	MAÑANA	FUTURE TENSE VERBS	Verb ending for future tense
	FUTURO	FUTURE CONDITIONAL VERBS	Verb ending for future conditional tense
	TU	YOU familiar	<i>tú, ti, tus, tuyo, tuyos, tuyas</i>
	EL	HE PRONOUN	<i>el, su, sus, suyo, tuyas</i>
	ESCUELA	SCHOOL	<i>saber, clase, matemática, lectura, ciencias naturales, arte, música</i>
	DADOS	GAMES	<i>jugar, aparejar, baraja, dados, tictacto</i>
	SOL	LIKE, DAY	<i>gustar, día, días</i>
	DINERO	MONEY	<i>tener, dinero, moneda, fería, nickel, dime, quarter</i>

Icon	Name	Category	Word Examples
	MASCARAS	FEELINGS	<i>sentir, sentimientos, feliz, triste, emocionado, enfadado</i>
	LIBRO	READING	<i>leer, libro, cuaderno, diario, revisto, cuento</i>
	MUSICA	MUSIC	<i>música, escuchar, album, rado</i>
	ZAPATO	SPORTS	<i>caminar, beisbol, básquetbol, ehercicio</i>
	MAPA	PLACES	<i>quedar, parte, partes, campo, ciudad, granja</i>
	ALTO	STOP	<i>parar</i>
	USTED	YOU formal	<i>usted, su, sus, suyo, suyos, suya, suyas</i>
	ELLA	SHE PRONOUN	<i>ella, su, sus, suyo, suya, suyos, suyas</i>
	NOSOTRAS	WE feminine	<i>nosotras, nuestro, nuestra, nuestros, nuestras</i>
	QUERER	WANT	<i>querer, enojar</i>
	FAMILIA	FAMILY	<i>familia, mama, papa, hermano, hermana, abuela, abuelo</i>
	PERRO	ANIMALS	<i>venir, animal, mascot, perro, gato, Sub-categories of animals</i>
	RELOJ	TIME	<i>comenzar, luego, listo, Days of the week, months</i>
	RECORDAR	REMEMBER	<i>recorder, cosa, cosas, importante</i>
	RANA	FROG, GO	<i>ir, lugar</i>
	ALBERCA	BODY PARTS	<i>nadar, cuerpo, Sub-categories of body parts</i>

Icon	Name	Category	Word Examples
	ARCO IRIS	COLORS and ART SUPPLIES	<i>pintar, rojo, anaranjado, amarillo, verde</i>
	AYUDAR	HEALTH and BODY	<i>ayudar, terapia</i>
	USTEDES	YOU plural formal	<i>ustedes, su, sus, suyo, suya, suyas</i>
	ELLOS	THEY PRONOUN	<i>ellos, su, sus, suyo, suyos, suya, suyas</i>
	OJO	VISION, SIGHT	<i>ver, vision, mirar, mirada</i>
	PENSAR	THINKING	<i>pensar, idea, inteligente</i>
	SALIDA	LEAVING	<i>salir, salida</i>
	TELEFONO	TALKING, SMALL APPLIANCES	<i>hablar, aparato, teléfono, celular</i>
	BUENO	GOOD, POSITIVE	<i>dar, permiso, bueno, Buena</i>
	CASA	HOUSE and BUILDINGS	<i>estar, casa, cuarto, sala, salon de recreo</i>
	AMOR	LOVE	<i>amar, amor, conmigo, querido</i>
	GENTE	PEOPLE	<i>gente, seguir, amigo, amiga, hombre, señorita</i>
	MAÑANATR	NATURE and VEHICLES	<i>subir, viaje, transportación, carro, troque</i>
	CERRAR	TOYS and CONTAINERS	<i>abrir, cosa, abierto</i>
	ELLAS	THEY feminine	<i>ellas, su, sus, suya, suyas, suyo, suyos</i>
	NO	NEGATIVE	<i>no</i>
	ORACIONES	SENTENCES	<i>decir, contar</i>

Icon	Name	Category	Word Examples
	SILLA	ASSISTIVE TECH	<i>sentar, discapacidad, tecnología asistiva</i>
	PAN	FOODS	Sub-categories of foods, <i>comer, comida</i>
	DIOS	HOLIDAYS and SHAPES	<i>hacer, días festivos</i>
	PARAGUAS	WEATHER	<i>clima, necesitar, soleado, lloviendo, nublado</i>
	BEBIDA	BEVERAGES	<i>bebida, tomar, té, café, leche, agua</i>
	POLICIA	WORK, WORKERS	<i>trabajar, maestro, asistente, doctor, terapeuta ocupacional</i>
	COCINA	KITCHEN	<i>cocina, volver, plato, platilo, tazón, taza</i>
	CAMA	FURNITURE and SLEEP	<i>dormir, sueño, muebles</i>

## Grammatical structure

The grammatical structure of Unity Español 84 is based on the following patterns.

**Pronouns:** The following pronouns are produced by twice activating the icon which represents the corresponding pronoun:

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### Possessive Pronouns:

Possessive pronouns are produced by activating the icon which depicts the corresponding prnoun followed by the icon NOMBRE and the desired possessive form.

Sequence	Word	English meaning
  	<i>mi</i>	my
  	<i>mío</i>	mine
  	<i>tu</i>	your
  	<i>tuyo</i>	yours
  	<i>su</i>	yours (his, hers)

Other words on the core display are generated using the following grammar labels:

**Interjections (Pirotécnicas):** This icon is used for interjections, or quick remarks/responses. it is anticipated that the individual using the Unity 84 español MAP will customize the interjections on the core display in accordance with his/ her age, gender social background, geographic factors and personal preferences.

Sequence	Word	English meaning
 	<i>Con permiso</i>	Excuse me
 	<i>¡Imaginate!</i>	Imagine that!
 	<i>muy bien</i>	Fine, very well

### More interjections

Sequence	Word	English meaning
 	<i>lo siento</i>	I'm sorry
 	<i>por favor</i>	please

These interjections are produced very rapidly.

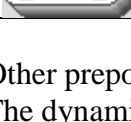
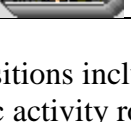
**Demonstratives (Mago):** This icon is activated by accessing the icon Mago (magician) and then a demonstrative term such as *ésto*, *eso*, or *esos*. (El mago señala.)

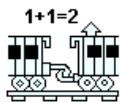
Sequence	Word	English meaning
 	<i>eso</i>	that
  	<i>esta</i>	this (fem.)
  	<i>estas</i>	these (fem.)
  	<i>ésto</i>	that
  	<i>estos</i>	these (masc.)

**Prepositions (Puente)** When the icon Puente is activated, prepositions appear in the core vocabulary display and in the activity row.

Sequence	Word	English meaning
 	<i>adelante</i>	forward, ahead
 	<i>adentro</i>	inside
 	<i>afuera</i>	outside
 	<i>arriba</i>	up
 	<i>contra</i>	against
 	<i>debajo</i>	under
 	<i>delante</i>	in front
 	<i>entre</i>	between
 	<i>excepto</i>	except
 	<i>frente</i>	forward, in front

Other prepositions include: *detrás*, *con*, *adentro*, *sobre*, *lejos*, *alrededor*, *enfrente*, *atrás* and *en*.  
The dynamic activity row for prepositions contains the following words: *aquí*, *allá*, *allí*, *por*, *de*, *atrás*, *encima*, *hacia*, *adelante*, *debajo*, *adentro*, *lejos*, *antes*, *cerca*, and the time words *antes* and *después*.

# Conjunctions (Tren)

This icon is used to produce conjunctions.

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### Questions (Pregunta)

The question mark is the beginning icon in question words, There is also a question activity row linked to the Question icon.

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### Adverbs (Adverbos)

The Adverb icon is the beginning of sequences for adverbs. A dynamic activity row contains adverb endings.

Sequence	Word	English meaning
 	<i>bien</i>	good, well
 	<i>siempre</i>	always
 	<i>tan</i>	so

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### Nouns (Gabinete Vacío)

This icon is the ending key for singular category nouns. Category nouns begin with an icon in core. Nouns that are members of the category appear in the dynamic activity row that is linked to the core icon. For example, the food nouns are in a dynamic activity row that opens when you select the PAN icon. The category noun *comida* appears on the GABINETE VACIO icon. The GABINETE LLENO icon is the ending key for plural category nouns. For example, the word *comidas* begins with PAN and ends with GABINETE LLENO.

Two hits on GABINETE VACIO adds an “s” ending to words. Two hits on GABINETE LLENO adds an “es” ending to words.

Here are the core nouns:

Sequence	Word	English meaning
 	<i>ropa</i>	clothing
 	<i>castigo</i>	punishment
 	<i>parte</i>	part
 	<i>clase</i>	class
  	<i>razón</i>	reason, right
 	<i>juego</i>	game
  	<i>premio</i>	prize

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## Nouns

Sequence	Word	English meaning
  	<i>salud</i>	health
 	<i>vision</i>	vision
  	<i>mirada</i>	look
 	<i>idea</i>	idea
  	<i>opinion</i>	opinion
 	<i>salida</i>	exit
 	<i>permiso</i>	permission
 	<i>aparato</i>	appliance, device
 	<i>casa</i>	house
  	<i>hogar</i>	home
 	<i>amor</i>	love
  	<i>vida</i>	life

## Nouns

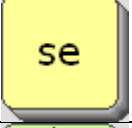
Sequence	Word	English meaning
 	<i>gente</i>	people
  	<i>amistad</i>	friendship
 	<i>viaje</i>	trip
 	<i>cosa</i>	thing
 	<i>discapacidad</i>	disability
 	<i>comida</i>	food
 	<i>religión</i>	religion
  	<i>Dios</i>	God
 	<i>clima</i>	weather
  	<i>lluvia</i>	rain
 	<i>bebida</i>	drink (n.)
 	<i>cocina</i>	kitchen

## Nouns

Sequence	Word	English meaning
  	<i>vuelta</i>	turn
 	<i>sueño</i>	sleep

## Dative Terms (modify an indirect object) and Articles

The following dative terms and articles appear in the core display. Many of these are single hits.

Sequence	Word	English meaning
	<i>a</i>	to, of, toward, at, on, in
	<i>que</i>	that
 	<i>se</i>	a reflexive pronoun indicator
 	<i>lo</i>	it
	<i>el</i>	the
	<i>la</i>	the
	<i>los</i>	the
	<i>las</i>	the
 	<i>de</i>	of

## Verbs

The following 41 verbs are primary verbs. The pattern for primary verbs is one hit on a core icon plus one hit on Mr. Action Man, “*monito de acción.*” This forms the infinitive of each primary verb.

Sequence	Word	English meaning
 	<i>abrir</i>	to open
 	<i>amar</i>	to love
 	<i>ayudar</i>	to help
 	<i>buscar</i>	to look for
 	<i>caminar</i>	to walk
 	<i>comenzar</i>	to start
 	<i>comer</i>	to eat
 	<i>decir</i>	to say
 	<i>dormir</i>	to sleep
 	<i>encontrar</i>	to find
 	<i>escribir</i>	to write

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Sequence	Word	English meaning
  escuchar	<i>escuchar</i>	to hear
  estar	<i>estar</i>	to be
  gustar	<i>gustar</i>	to like
  hacer	<i>hacer</i>	to make
  ir	<i>ir</i>	to go
  jugar	<i>jugar</i>	to play
  leer	<i>leer</i>	to read
  nadar	<i>nadar</i>	to swim
  necesitar	<i>necesitar</i>	to need
  pensar	<i>pensar</i>	to think
  pintar	<i>pintar</i>	to paint
  poner	<i>poner</i>	to put

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Sequence	Word	English meaning
 pregunta	<i>preguntar</i>	to ask
 quedar	<i>quedar</i>	to stay
 querer	<i>querer</i>	to want
 quitar	<i>quitar</i>	to take away
 recordar	<i>recordar</i>	to remember
 saber	<i>saber</i>	to know
 salir	<i>salir</i>	to leave
 seguir	<i>seguir</i>	to follow
 sentar	<i>sentar</i>	to sit
 sentir	<i>sentir</i>	to feel
 ser	<i>ser</i>	to be
 subir	<i>subir</i>	to climb, to go up

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Sequence	Word	English meaning
 	<i>tener</i>	to have
 	<i>tomar</i>	to drink
 	<i>trabajar</i>	to work
 	<i>venir</i>	to come
 	<i>ver</i>	to see
 	<i>volver</i>	to turn

### Secondary Verbs

The following 31 words are secondary verbs. The pattern for secondary verbs is two hits on a core icon plus Mr. Action Man, “*monitor de acción*”.

It should be noted that many of the secondary verbs are either synonyms or antonyms with respect to primary verbs created using the same core icons.

Sequence	Word	English meaning
  	<i>aprender</i>	to learn
  	<i>bajar</i>	to get down, to go down, to come down
  	<i>cambiar</i>	to change

## Secondary Verb Examples

Sequence	Word	English meaning
  	<i>cerrar</i>	to close
  	<i>cocinar</i>	to cook
  	<i>comprar</i>	to buy
  	<i>creer</i>	to believe
  	<i>cuidar</i>	to take care
  	<i>descansar</i>	to rest
  	<i>encontrar</i>	to find, to meet
  	<i>enojar</i>	to make angry
  	<i>estudiar</i>	to study
  	<i>ganar</i>	to get, to win
  	<i>haber</i>	to have (auxiliary)
  	<i>hablar</i>	to talk

## Secondary Verb Examples

Sequence	Word	English meaning
  	<i>llegar</i>	to arrive
  	<i>llover</i>	to rain
  	<i>manejar</i>	to drive
  	<i>meter</i>	to put
  	<i>cantar</i>	to sing
  	<i>olvidar</i>	to forget
  	<i>pasar</i>	to pass
  	<i>pedir</i>	to ask, to request
  	<i>poder</i>	to be able, can
  	<i>reunir</i>	to come together
  	<i>terminar</i>	stop
  	<i>traer</i>	to bring, to carry

## Secondary Verb Examples

Sequence	Word	English meaning
  	<i>llevar</i>	to wear, to carry
  	<i>vivir</i>	to live

## Verb conjugations

In the third row of the Unity 84 Español overlay, there are labels which represent verb tenses:  
:

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**Verb Examples:**

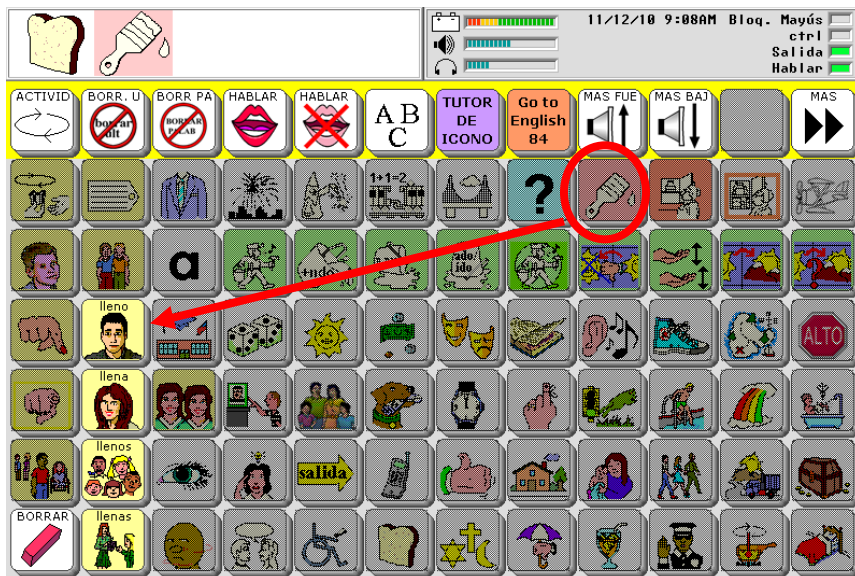
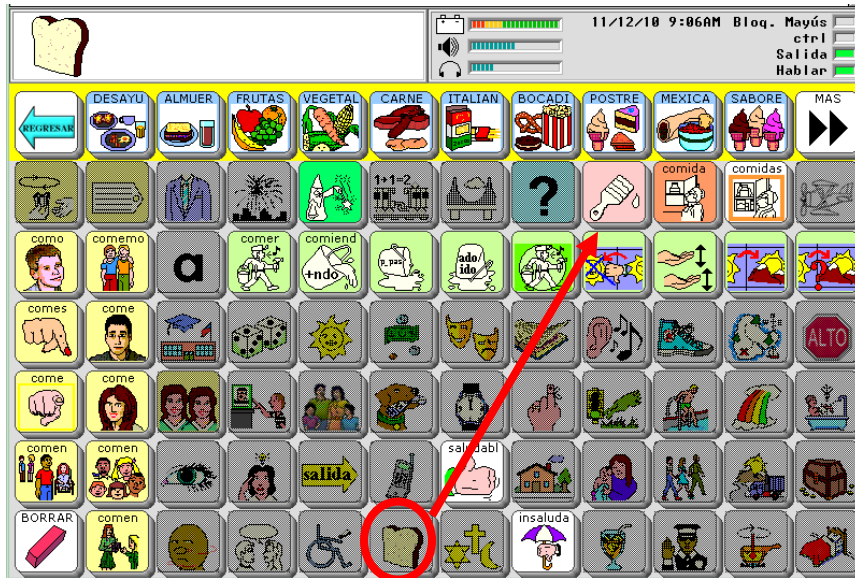
Sequence	Word	Significado en ingles
 	<i>comer</i>	to eat
 	<i>tomar</i>	to drink
  	<i>tomo agua</i>	I drink water
 	<i>comiendo</i>	eating
 	<i>tomando</i>	drinking
 	<i>como</i>	I eat
  	<i>comió</i>	you (formal) ate
  	<i>comió</i>	he ate
  	<i>comió</i>	she ate
  	<i>ha comido</i>	he had eaten
  	<i>ha tomado</i>	she had drunk
  	<i>comía</i>	he was eating, used to eat, ate

Sequence	Word	Significado en ingles
  	<i>comiera</i>	you (formal) ate, were eating
  	<i>coman</i>	you (formal, plural) eat, are eating
  	<i>tomen</i>	they drink, are drinking
  	<i>comeremos</i>	we will eat

## Adjectives

Adjectives are found by choosing a category key, such as the PAN icon for food, then selecting the pink paintbrush, and finally an ending key to indicate gender and number.

### Adjective Examples

Sequence	Word	Significado en ingles
  	<i>lleno</i>	full
  	<i>llena</i>	full
  	<i>llenos</i>	full
  	<i>llenas</i>	full
  	<i>corto</i>	short
  	<i>corta</i>	short
  	<i>cortos</i>	short
  	<i>cortas</i>	short
  	<i>largo</i>	long
  	<i>larga</i>	long
  	<i>largos</i>	long
  	<i>largas</i>	long

## Adjective opposites

In addition, you can find more adjectives and adverbs of neutral gender by choosing a category key, and then either the thumbs up key or the thumbs down on the umbrella

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## Use the Icon Tutor to find more words

You can find any word in the Unity 84 Spanish MAP by using the icon tutor. Select the key marked TUTOR DE ICONO, and then follow the prompts on the screen.

**You can find the documentation for English Unity 84 Sequenced in the Vantage Lite Unity Manual. See the CD that came with your bilingual Vantage Lite.**

## **Adding your customized area to Spanish bilingual**

**Very important! If you are upgrading your device by inserting a new memory card for Spanish, do these steps first with your old card still in the device.**

If you already have an English speaking user area customized in Unity, you may want to add this area to your Spanish Vantage Lite. For example if you are speaking with Unity 60 Sequenced and you have your own user area, you can use it instead of one of the English areas in the bilingual Vantage or Vanguard. Here are the directions for adding in your customized area:

### **Transfer the memory of your English-speaking area**

1. Put a USB drive into your device.
2. Go to the ToolBox, then to the Transfer Memory Menu.
3. Select Save One User Area.
4. Select your customized user area.
5. Follow the prompts on the screen to save your area to the USB drive.

**Next put in the new Spanish memory card and reset your device by holding down the on/off key for 20 seconds.**

1. Put the USB drive into your device.
2. Go to the Toolbox, then to the Transfer Memory Menu.
3. Select Load One User Area.
4. Select one of the English areas on the left. Choose one that you are not going to be using because it will be erased.
5. Select your custom area that you saved to the USB drive.
6. Follow the prompts on the screen to complete the memory transfer.

### **Make a link between your new Spanish area and your custom area.**

1. Find the orange key in the device tools activity row that says Go to English .
2. Select the Set-up key, and then the orange key.
3. Select Spell Message or Define Key Function.
4. Select Clear Display.
5. Select Insert Tool and then Switch User Area.
6. Select your custom user area.
7. Change the label on the key if you want.
8. Select OK, and OK again.
9. Use the orange key to go to your custom area.
10. Find a key in the device tools activity row where you can make a link.
11. Use the directions above to make a link back to your new Spanish user area.

**If you need help with these steps, please call PRC Service at 1-800-262-1990.**