

# Vantage™ Lite USB Software Update

for 5.05 and higher software

---

➤ **The update *may* take up to three hours, so you need to prepare for this.** You cannot use the device during the update.

➤ **Read the instructions before beginning the update.** You may want to print this page and then proceed to the update.

➤ **You will need** your Vantage Lite and the battery charger that came with the device.

The system software you are currently using in your device should be a **lower version** than the version you want to download. To check your current version, go to the Vantage Lite Toolbox. Open the **MAINTENANCE MENU**. Look at the "System Information" box to check your current software version.

## On Your Computer

1. Go to [www.prentrom.com](http://www.prentrom.com)
2. Click on "Downloads." Click on "Vantage Lite." Click on "Software Updates".
3. Download the update file to your computer desktop. ➤ If there is more than one file, **download each file** to your computer desktop.
4. Plug the USB flash drive into the USB port on your computer.
5. Copy the update file to your USB flash drive.
6. Remove the USB flash drive from your computer port and keep it nearby.

## On Your Device

1. Plug your USB flash drive into a USB port on the right side of the VTL.
- If you have made changes to your device vocabulary that you want to keep, use the **TRANSFER MEMORY MENU** in the Vantage Lite Toolbox to save the vocabulary to your USB flash drive before starting the update.
2. Turn the Vantage Lite ON and plug in the battery charger.
3. **Hold the ON/OFF button for 10 seconds.** The update will now begin.
4. When the update is finished, remove the USB flash drive from the device.
5. If you saved vocabulary earlier, use the **TRANSFER MEMORY MENU** to load it back into the device now.
- **Note:** If you load old vocabulary, you will lose any MAP fixes or MAP changes that came with the new software.
6. Remove the USB drive from your VTL and put it in a safe place. You may also unplug the battery charger if you wish.