

# ECO<sup>™</sup> 2 USB Software Update

---

- **Read the instructions before beginning the update.**  
You may want to print this page and then proceed to the update.
- **To download and install this update you will need:**  
Your ECO2 and the ECO2 battery charger.
- The system software you are currently using in your device should be a **lower version** than the version you want to download. To check your current version, go to the Toolbox. Open the **MAINTENANCE MENU**. Look at the "System Information" box to check your Current Software Version.

## On Your Computer

1. Go to [www.prentrom.com](http://www.prentrom.com)
2. Click on "**Downloads.**" Click on "**ECO2.**" Click on "**Software Updates.**"
3. Click on the update you want and download it to your **computer desktop.**
4. Plug a USB flash drive into the USB port on your computer.
5. Copy the update file to your USB flash drive.
6. Remove the USB flash drive from your computer.

## On Your ECO2

1. Plug your USB flash drive into a USB port on the right side of the ECO2.
- **Note:** ECO2 automatically performs a **Reset to Factory Defaults** during the update process. **This erases your vocabulary.** If you have made changes to your vocabulary that you want to keep, use the **TRANSFER MEMORY MENU** in the Toolbox to save the vocabulary to a USB flash drive **before** you start the update. Click on or go to the following link for directions: <http://www.prentrom.com/support/article/1030>
2. Turn **On** the ECO2 and plug in the battery charger.
3. Go to the Toolbox and open the **MAINTENANCE MENU**.
4. Select the **Update Software** option. Make sure the battery charger is plugged in.
5. Answer **YES** to continue.
6. Answer **YES** to continue
7. When the update is finished, remove the USB flash drive from the device.
8. If you saved vocabulary earlier, use the **TRANSFER MEMORY MENU** to load it back into the device now.
- **Note:** If you load old vocabulary, you will **lose** any MAP fixes or MAP changes that came with the new software.
9. Remove the USB drive from your device and put it in a safe place. You may also unplug the battery charger if you wish.