



## Instructions to Load the new SpringBoard IntelliTools Activity Row


Load the **SB IT Activity Row** MTI file into a User Area that you are willing to erase. To do this, select the TRANSFER MEMORY MENU and choose LOAD ONE USER AREA.

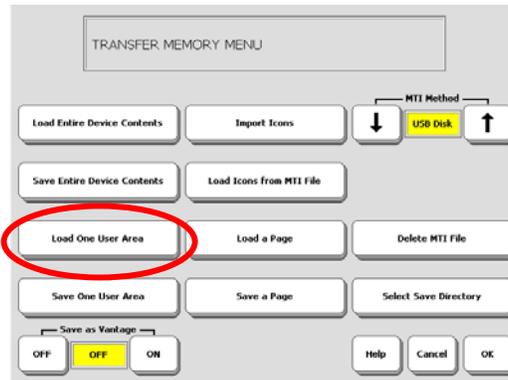


Follow the prompts to load the **SB IT Activity Row** MTI. Then go into the CHOOSE OVERLAY MENU and select SWITCH USER AREA Switch User Area. Switch to the area that has the **IT Activity Row**.

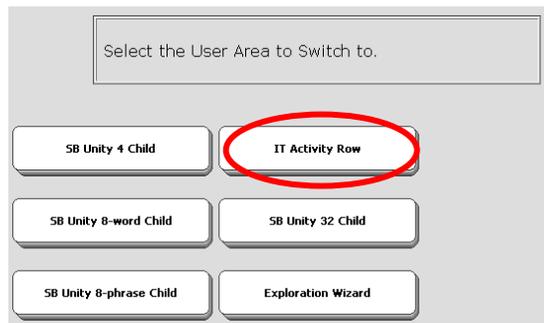

Now select Copy Pages and Activities

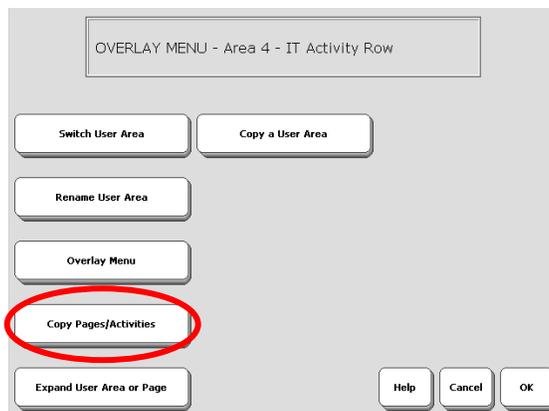


## Select Copy Selected Activity

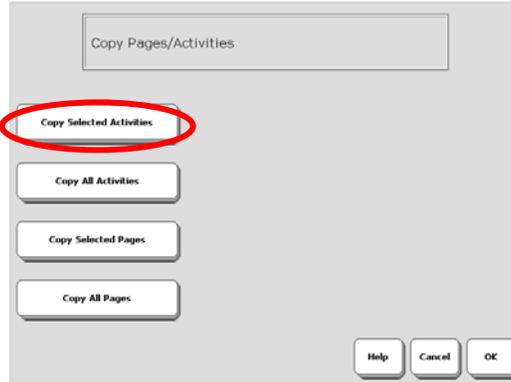

Copy the activity to SB Unity 32.



There is only one activity to copy, select it, and choose OK. Answer YES and select OK.

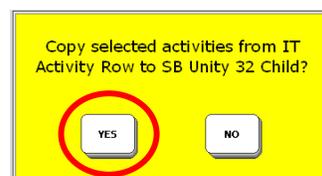

Say YES to replace the IntelliTools Activity



Select OK and then switch to the user area with SB Unity 32.