

USB Flash Drive Software Update Instructions For SpringBoard Lite

- ➔ **Read the instructions before downloading the update.** You may want to print this page and then proceed with the download.
- ➔ **This update has been known to take up to 30 minutes.** There are instances where it may take longer. Please be patient and allow for the time involved.
- ➔ **To receive any MAP fixes or MAP changes in this new software** you must either load the default MAPS into the User Areas on your SpringBoard Lite or reset your device to Factory Defaults (in MAINTENANCE menu) at the end of the download procedure.

TO DOWNLOAD

- 1: If you want to use the vocabulary currently in your device, back it up now before you proceed with the download.
- 2: Plug your USB flash drive into the USB port on your computer. (You will need a lot of free space on your flash drive.)
- 3: Download the update file to your computer desktop.
- 4: Copy the file to your USB drive.
- 5: When the file has been copied, unplug your USB flash drive from the computer and plug it into the USB port on the side of your SpringBoard Lite.
- 6: Turn on your SpringBoard Lite and plug in its charger.
- 7: Reset the device by holding the ON/OFF button for 10 seconds. The update will now begin.
- 8: When the update is finished, remove the USB flash drive from the device.
- 9: Unplug the charger once the device is running the new software.
- 10: Reset the device to Factory Defaults using the MAINTENANCE menu.
- 11: If you saved your vocabulary, you can now load it back into the device. This means you will **lose** any MAP fixes or MAP changes that came with the new software.