

## **SpringBoard USB Flash Drive Software Update Instructions**

**\*NOTE:** please read the instructions before downloading the update. You may want to print this page then proceed with the download.

**\*NOTE:** To download and install this update you need: SpringBoard Plus and 3.0x system software. To get new fixes to maps you will need to either load the default maps or reset the device to the factory defaults.

**\*\*\* This software update will only work on a IBM compatible PC \*\*\***

**\*\*\* No MAC version is available \*\*\***

**Step 1:** Backup your device vocabulary.

**Step 2:** Plug in your USB flash drive into the computer. (8MB free space is needed on the Flashdrive)

**Step 3:** Download the update file to your desktop.

**Step 4:** Copy the file to your USB flash drive.

**Step 5:** Once the file is copied, unplug the USB flash drive from the computer and plug it into your SpringBoard.

**Step 6:** Turn on your AAC device and plug in the charger.

**Step 7:** Reset the device by holding the ON/OFF button for ten seconds. The update will now begin.

**Step 8:** Once the update is done you can remove the USB flash drive.

**Step 9:** Unplug the charger once the device is running the new software.

**Step 10:** Reset the device to factory defaults.

**Step 11:** Load the vocabulary back into the device.