

Vantage-Vanguard USB Flash Drive Software Update and Instructions

***NOTE:** Please read the instructions before beginning the update. You may want to print this page then proceed.

***NOTE:** To download and install this update you need: Vantage Plus and 4.0x system software or Vanguard Plus (serial number 2100 and up) and 4.0x system software or an updated Vanguard that has MP3 player and USB keyboard mouse functionality. To get new fixes to maps you will need to either load the default maps or reset the device to the factory defaults.

***** This software update will only work on a IBM compatible PC *****

***** No MAC version is available *****

Step 1: Backup your device vocabulary.

Step 2: Plug in your USB flash drive into the computer. (8MB free space is needed on the Flashdrive)

Step 3: Download the update file to your desktop.

Step 4: Copy the file to your USB flash drive.

Step 5: Once the file is copied, unplug the USB flash drive from the computer and plug it into your Vantage/Vanguard.

Step 6: Turn on your AAC device and plug in the charger.

Step 7: Reset the device by holding the ON/OFF button for ten seconds. The update will now begin.

Step 8: Once the update is done you can remove the USB flash drive.

Step 9: Unplug the charger once the device is running the new software.

Step 10: Reset the device to factory defaults.

Step 11: Load the vocabulary back into the device.