

# **New Software Features**

## **In Vantage™/Vanguard™ with 4.05 Software**

---

### **Change the Activity Row Background Color**

In the Toolbox, select the MODIFY ACTIVITY menu. Select the "Activity Background Color" option. Select a color for your Activity Row. The color remains the same for all Activity Rows until you change it. **Note:** You cannot access the Activity Background Color key using the SETUP key.

### **Block Scan is now available**

In the SELECTION TECHNIQUE MENU, select either "1-Switch" or "2-switch" as the "Selection Technique". In the "Scan Direction" option, select either "Quarter Row/Column" or "Quarter Column/Row".

### **Adult Acquired Overlays are now available**

**IMPORTANT!** Loading Adult Acquired requires the use of all six User Areas. If you have previously customized any User Area and want to save the customization, do a memory transfer first to save the area(s).

Open the CHOOSE OVERLAY MENU. Select the "Overlay Menu" option. Select "Load Adult Acquired (All Areas)". To return to default Unity overlays, go to the MAINTENANCE MENU and select "Reset to Factory Defaults".

### **IntelliTools™ Button Maker Tool is an Insertable Tool**

If you are programming keys for IntelliTools activities, you can now use "Insert Tool", then choose the "IntelliTools Button Maker" tool to use in macros that work with IntelliTools software.

### **New Phone Options "Import Custom Ringtones" and "Caller ID" are now available**

If you have purchased the optional Aircard for your device, use the "Phone Configuration Menu" in the MAINTENANCE MENU to import a new ringtone from an MP3 file on your USB flash drive. "Caller ID" automatically shows the caller's phone number when you have the Dial Pad open.

### **TASH™ Phone IR Commands are now in the IR SETUP MENU**

If you purchase a TASH infrared phone, you can use the pre-programmed commands on the TASH phone pages in all Unity areas and in Word Core.