

Directions for loading Education Software pages and activity rows

1. Copy the files for Education Software onto your USB drive.
2. Put the USB drive into the device.
3. In the device ToolBox select Transfer Memory Menu.
4. Select Load One User Area.
5. Select a User Area that you are not using. All of the contents of this area will be erased during the transfer.
6. At the yellow warning flag, select YES.
7. Select the file of Education Software that you want from your USB drive.
8. When the transfer is finished, select OK, and then OK at the bottom right to return to the device ToolBox.
9. Select Choose Overlay Menu.
10. Select the area where you loaded the Education Software (84 ES or 45 ES).
11. Select Copy Pages/Activities
12. Select Copy All Activities.
13. Select your User Area and the activities will be copied there.
14. In the Choose Overlay Menu select Copy Pages/Activities.
15. Select Copy All Pages.
16. Select your User Area and the pages will be copied there.
17. When you see the yellow warning flag, select YES.

Directions for linking Education Software pages and activity rows

1. In the Choose Overlay Menu select Switch User Area.
2. Select your User Area, and then OK.
3. Select Modify Activity.
4. Select Education Software, and then Show. This puts the key for Education Software into the main Activity Row.
5. Finish by moving the Education Software activity row where you want it.
6. In Unity 84, link the Education Software page to a key in the school activity row.
 1. Select the School Activity Row.
 2. Select More Items. You will see blank keys.
 3. Select the device Set Up key.
 4. Select a blank key in the School Activity Row.
 5. Select Spell Text for Message.
 6. Select Insert Tool.
 7. Use the More Items key at the bottom of the page to find Set Page.
 8. Select Set Page.
 9. Select Kurzweil.
 10. Use the same directions for setting in WYNN, TestTalker, Word, and Outlook.
 11. Finish each key with an icon and a label.

7. In Unity 45, you can set the Education Software pages into the Page Page.